

New Core + Advanced Features



Introduction




Dr. Daniel G. Stewart
PT, DPT

 [dr.danielstewartdpt](#)



Agenda

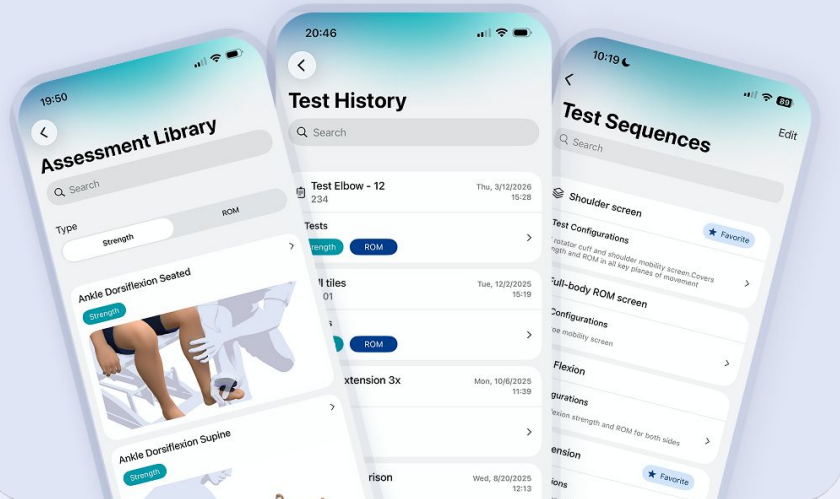

- Overview of new ActivForce Updates
- Core updates:
 - Account Management
 - Subject Details
 - Dashboard Test Reports
- Advanced features:
 - Test Sequences
 - Assessment Library
 - Test History
- Live demo of each feature in the app and on the web dashboard
- Open Q&A

Overview

Difference between

CORE

&

ADVANCED

- ActivForce Advanced is an optional plan that adds three workflow tools on top of the existing Core experience.
- Core stays free, keeps getting updated, and nothing changes for users who don't want Advanced.
- Advanced is \$20 per month, cancel anytime, and every new user gets a free trial to see if it fits their workflow.

Core Upgrades

Upgrades to the Free Core version

- Account Management
- Subject Details
- Dashboard Test Reports

Account Management



Test Reports

Search...

Track Progress

Saturday, February 14, 2026

Test Series 35

09:32 AM

Strength Right Shoulder Flexion

Time to Peak Force 3.15 s

Show Report

Knee Flexion Seated

EDC

07:56 AM

Strength Right Knee Flexion Seated

Time to Peak Force 1.53 s

Strength Left Knee Flexion Seated

Time to Peak Force 2.94 s

Strength Right Knee Flexion Seated

Peak Force 49.6 lb

Time to Peak Force 1.65 s

... 1 test hidden

Show Report

Knee FLEXION Seated

EDC

07:54 AM

Strength Right Knee Flexion Seated

Peak Force 44.5 lb

Time to Peak Force 2.60 s

Strength Left Knee Flexion Seated

Peak Force 36.1 lb

Time to Peak Force 4.46 s

Strength Right Knee Flexion Seated

Peak Force 43.0 lb

Time to Peak Force 2.82 s

Show Report

Account Management

- Users can now change their email independently without contacting support
- Users can now add company name to account


Subject Details



Knee Flexion Seated

Tuesday, January 20, 2026 at 07:56 AM

Subject Details

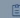 Edit Subject Details

Identifier	EDC	Gender	—
Weight	—	Dominant Side	—
Age	—		

Collapse All Expand All

#1 Strength Knee Flexion Seated (Left/Right)

Knee Flexion Seated

Subject Details

Identifier: EDC


Weight: 185 lb

Age: 40

Gender: Male

Dominant Side: Unspecified

Save

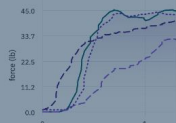

Show All

Graph Illustration

Right 3


Peak Force Time to Peak Force Average Peak Force

	Rep. 1	Rep. 2	Rep. 3
Left	54.1 lb	—	—
Right	41.2 lb	49.6 lb	40.6 lb

Subject Details

- Add subject gender, age, dominant side, and weight
- When weight is filled out, the system calculates force to weight ratio


Dashboard Test Reports

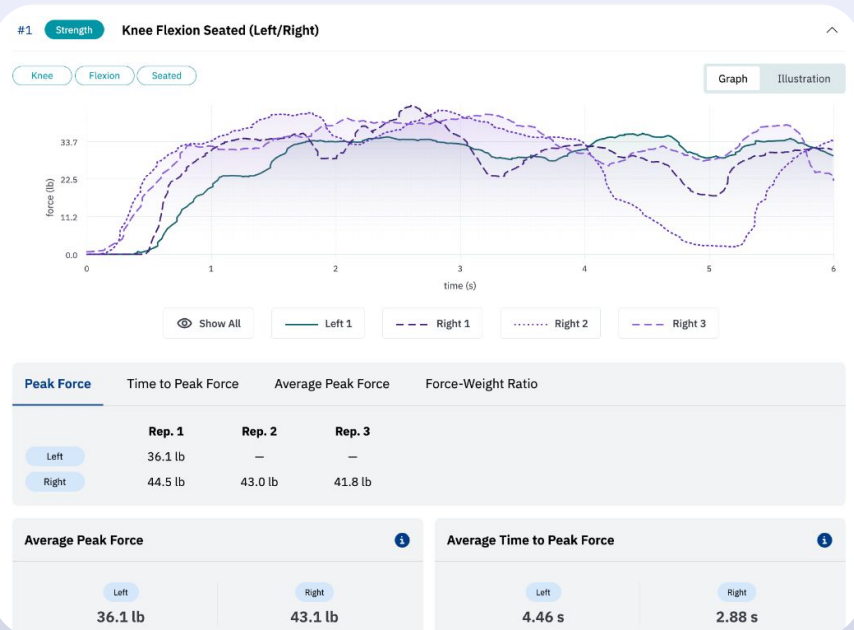


Dashboard Test Reports

- Enhanced visibility of metrics with left-to-right comparisons and organized tiles
- Ability to toggle between peak force and average peak force
- If subject weight is added, the average force-weight ratio is calculated
- Info button shows exact calculations for each measurement


Old Version

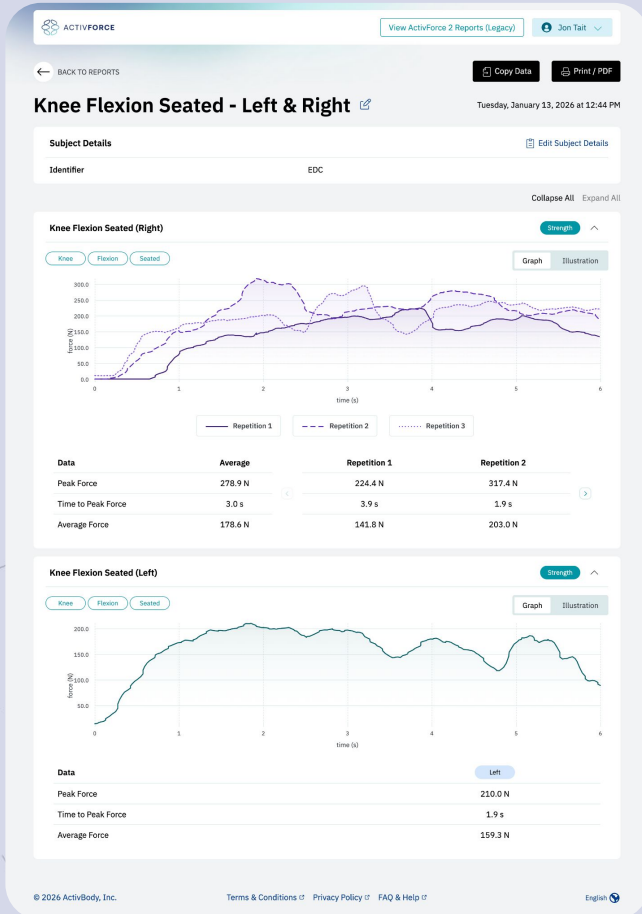


Core Upgraded

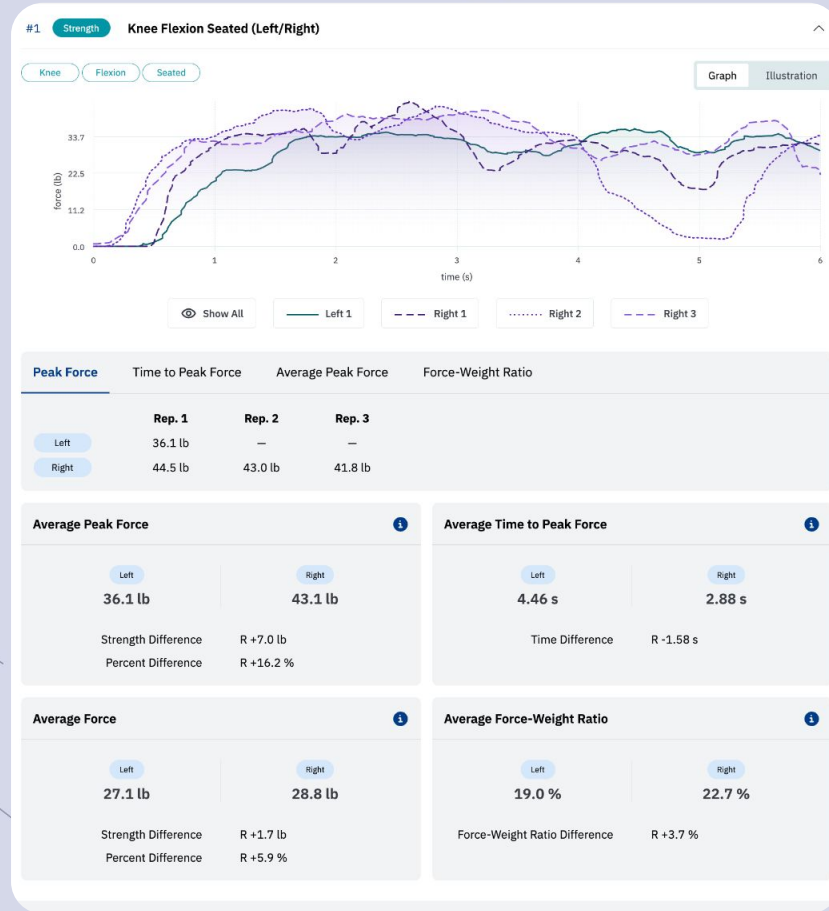

ActivForce Advanced

- Test Sequences
 - Assessment Library
 - Test History
- 
- The background features several decorative, dashed white lines that form smooth, flowing curves across the lower and right portions of the slide.


Advanced Features

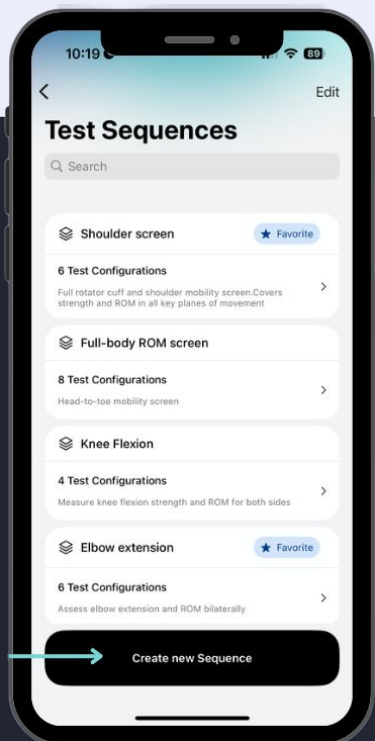

Test Sequences

- Build custom test sequences
 - Combine multiple tests into a single, repeatable flow.
- Save to Favorites for quick access from the home screen.


Advanced Features

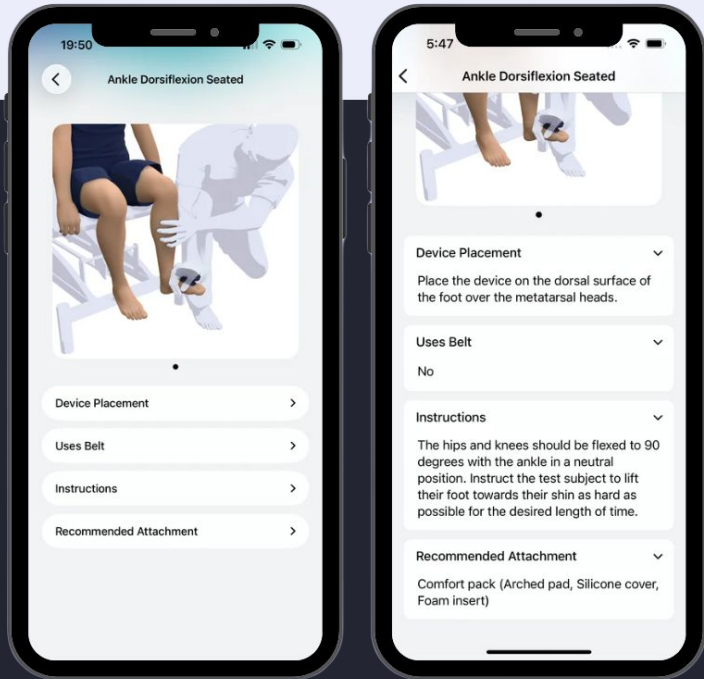

Assessment Library

- Testing protocols and visual references directly inside the app.
- Verify positioning or technique mid-session without leaving the workflow.

Advanced Features

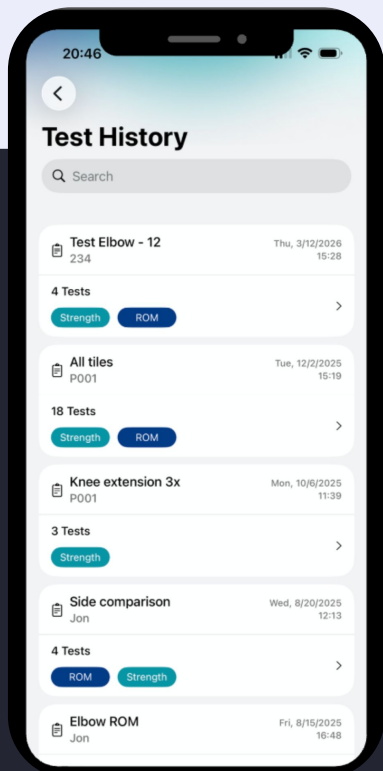

Test History

- Provides access to all past test sessions with full details
- Users can repeat test configurations from past sessions
- Users can create new sequences from past tests

ActivForce Advanced

Price	\$20 per month
Free Trial	30 Days Free Trial - Standard 60 Days Free Trial - Sign up before May 15, 2026
How to Activate	Log into the ActivForce app or dashboard → Manage Plan → select Advanced.

ActivForce Advanced

Cancellation	Cancel anytime. Access will continue until the end of the billing period.
Core access after cancel	Completely unchanged. Device, data, reporting, sync, offline testing — all intact.
Advanced data after cancel	Test Sequences, Assessment Library, and Test History are no longer accessible once the plan ends.
Payment	All major credit and debit cards, Apple Pay, and Google Pay. Multiple currencies supported.

Questions?

